

INTEGRATION GUIDE

GE Healthcare
CR1500



VERSION 01
RELEASE DATE: MAY 2018

www.codecorp.com

code®
Expect More.

Configuring CR1500 Barcode Readers

To configure a CR1500 to be dedicated to a patient room or workstation on wheels, scan the following barcode:



M20219_02

CR1500 Feedback Settings

Scan the preferred feedback settings:



M20329_01

Beep On
Vibrate On



M20331_01

Beep On
Vibrate Off



M20330_01

Beep Off
Vibrate On

CR1500 Reader Settings



M20111_01

Reset to factory defaults

Firmware Version Information

Scan the following barcodes to identify firmware versions:



M20361_01

Reader information